

T Thinking g

VIP
inserto[®]
SEAT


S E A T



2 A Teschio e denti, 1489.

RL 19058 v (B. 41 r; K/P 42 v).

Il vacuo della cassa dell'occhio, e 'i vacuo dell'osso sostenitore della guancia, e quello del naso e della bocca sono d'eguale profondità e terminano sotto il senso comune [sede centrale delle sensazioni] per linea perpendicolare. E ciascuna d'esse vacuità ha tanto di profondità quant'è la terza parte del volto dell'omo, cioè dal mento a capegli.

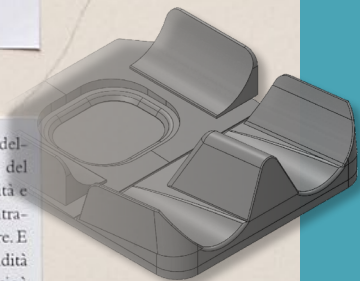

A NEW SOLUTION
INDIVIDUALLY CUSTOMISABLE

A NEW SOLUTION INDIVIDUALLY CUSTOMISABLE Construction kit of the individually customisable seat Perfect reproduction of anatomical shapes




Why Vip ?



- LOADS DISTRIBUTION
- LATERAL CONTAINMENT
- SCHIAL, TROCANTHERIC, COCCYGEAL AND SACRAL SUSPENSION



STRUCTURAL MODULAR KIT WITH DIFFERENT DENSITIES

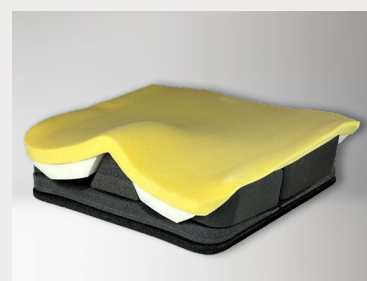

Free Shape PADDING SYSTEM



INTERNAL REMOVABLE COVER

CLINICAL INFORMATION / DESTINATION OF USE

The Pelvis Positioning System Inserto Seat Vip is recommended preferably, but not exhaustively for users with limited mobility with moderate deformities who need discrete postural containment and stabilisation, in particular:

- Those who have leg length discrepancies, wind-swept, pelvic obliquity, pelvis rotation, anterior and posterior pelvis tilt, adduction and abduction of the legs, of medium/severe entities;
- Those who need to stabilise and/or align ischium;
- Those who need ischial, trochanteric, coccygeal and sacral suspension;
- Those having high risk to develop decubitus sores;
- Those who need to balance and contain legs/pelvis/rachis/ relationship system .

The pelvis positioning system has to be used with any supporting/wheelchair base properly fixed through suitable gripping systems.

PTS Pelvic Total Support

TECHNICAL FEATURES

Each Positioning System Inserto Seat Vip includes:

- Structural kit with different densities composed by a flat base of construction and 8 closed-cell polyethylene inserts, modular and configurable, useful to support or correct the posture of the pelvis during the life time use;
 - Padding washable with gentle detergent composed by different layers of foam:
 - an upper memory foam layer for higher protection from the risk of decubitus sores,
 - a lower flexible and elastic polyurethane foam layer;
 - Internal padding cover made of PU/PES combined material with the following properties: antibacterial, non-cytotoxic, non-irritating, resistant to virus penetration, anti-fungal, fireproof, biocompatible, OEKO-TEX® class I certified, latex-free, breathable and waterproof;
 - External cover with the following properties: ventilated, flame retardant, antibacterial, latex free with low risk of skin irritation and commonly used in medical devices.
- Both covers are removable and can be washed in warm water with a gentle detergent. Drying machines are not recommended. The material does not require the use of disinfectants.

SPARE PARTS


- Flat base of construction
- Internal Removable Cover
- Padding
- External Removable Cover

WARNINGS

The Positioning System **Inserto Seat Vip**, in its design integrity and numerical and dimensional completeness of the supplied components (Structural Kit, Padding, Internal Removable Cover and External Removable Cover) can be easily adapted to sizes/morphology/deformities of the user. This kind of operations make the commissioning **referable to a serial device**.

Alternatively

The commissioning of the Positioning System **Inserto Seat Vip**, deprived or modified, even partially, of its design integrity of numerical and dimensional completeness of the supplied components, built as per written prescription of a professional operator in function of the anatomy/morphology/deformities of the user, through the measurement detection and direct trials, can be **referable to a custom device**.

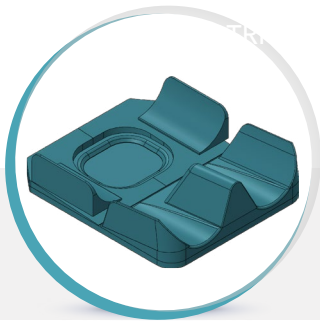

Inserto Vip

8 MODULAR AND CONFIGURABLE INSERTS

ANTIBACTERIAL, FIREPROOF AND VENTILATED **DOUBLE COVER**


° Each postural unit is ordered based upon the size of mobility/wheelchair device.

INSERTO VIP SIZES TABLE	PEDIATRICO								ADULT								SIZES (cm)	
	3232	3236	3434	3438	3638	3642	3840	3845	4040	4045	4242	4245	4250	4545	4550	4848	4850	5050
MODEL	3232	3236	3434	3438	3638	3642	3840	3845	4040	4045	4242	4245	4250	4545	4550	4848	4850	5050
Effective width (cm)	32	32	34	34	36	36	38	38	40	40	42	42	42	45	45	48	48	50
Max. width achieved with adaptation (cm)	34	34	36	36	38	38	40	40	42	42	44	44	44	47	47	50	50	52
Min. width achieved with removal of material (cm)	30	30	32	32	34	34	36	36	38	38	40	40	40	43	43	46	46	48
Effective length (cm)	32	36	34	38	38	42	40	45	40	45	42	45	50	45	50	48	50	50
Max. length achieved with adaptation (cm)	34,5	38,5	36,5	40,5	40,5	44,5	42,5	47,5	42,5	47,5	44,5	47,5	52,5	47,5	52,5	50,5	52,5	52,5
Min. length achieved with adjustment through seat notches and removal inserts (cm)*	29	33	31	35	35	39	37	42	37	42	39	42	47	42	47	45	47	47
Min. and max. range of anterior height of flat base of construction (5)+ leg levelling insert (1) (cm)**					3,5 / 4,5				** height referred to models 3232 e 5050									
Min. and max. range of posterior height of flat base of construction (5) + pelvic containment inserts (4) (cm)**					9 / 13				** height referred to models 3232 e 5050									

Weight of positioning system (min/max): 0,9 kg/3 kg

Max. load referred to model 5050: 150 kg


*possible and further reductions due to removal of material

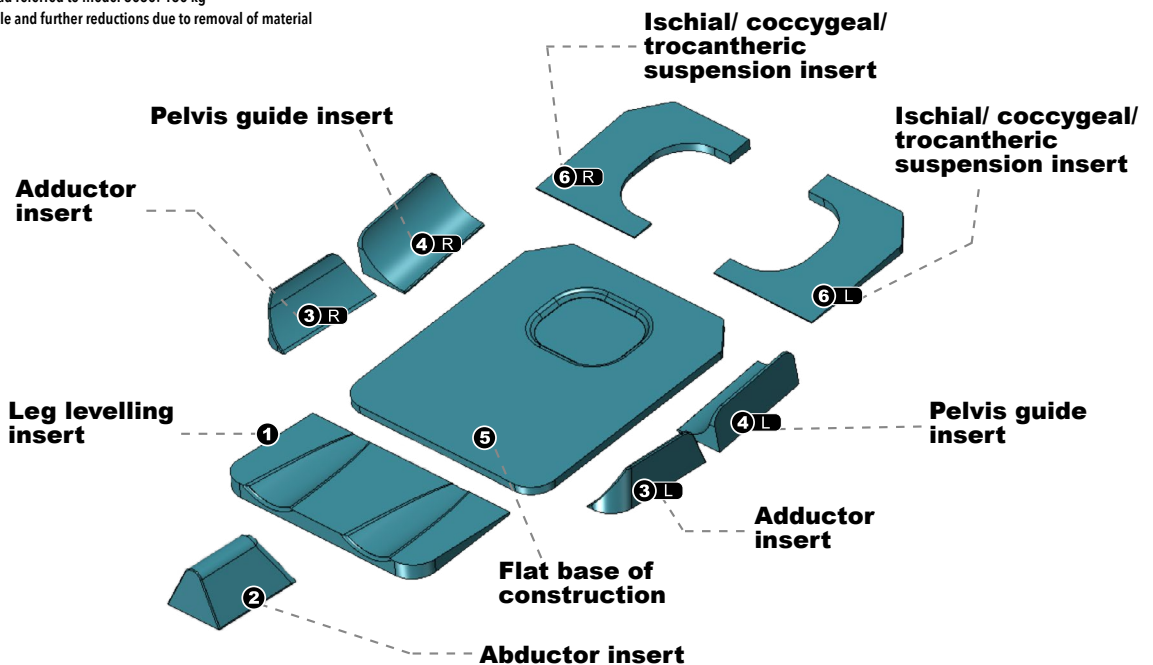


VIP

VIP
inserto[®]
SEAT


Customisations of Vip

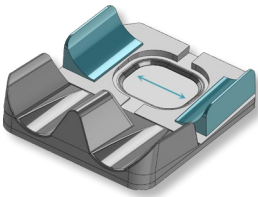


ADJUSTMENT OF TROCHANTERIC
SUPPORT PREVENTING ISCHIAL LOADS

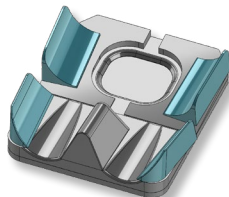


FIXED
ADDUCTION

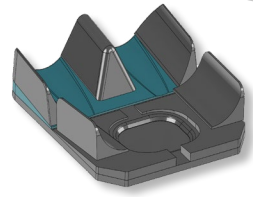


ANTI-THRUST
OF THE PELVIS

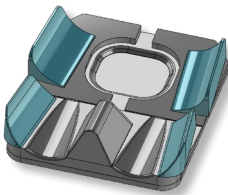


FIXED
ABDUCTION

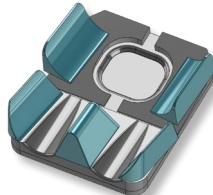

WIND SWEEP

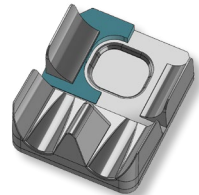

FIXED ANGLE

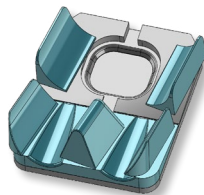

PELVIC ROTATION

INDIVIDUALLY CUSTOMISABLE

Construction kit of the individually customisable seat

THE INNOVATIVE SOLUTION

A range developed to answer the individual postural need (as per prescription) and numerous pathologies throughout the whole life cycle of use.

In order to maintain its features it is necessary to periodically check the conservation and wear of all the parts composing the device as well as all hook and loops stripes. The positioning system seen the sizes and weight results easily transportable.

1 MAN/WOMAN ADAPTABILITY

Finally a solution that makes the difference with a fully personalised fit.

2 ADAPTABILITY TO THE INDIVIDUAL MORFOLOGY AND ANATOMY


Unlimited possibilities to realize the customised postural solution.

3 PTS PELVIC TOTAL SUPPORT

A perfect fit to anatomy of the pelvis and related body segments, encouraging an active response of the musculo-skeletal system. The distribution of the loads provided by the PTS reduces the mechanical stress on the deep tissues and significantly aids the prevention of decubitus ulcers.

4 FREE SHAPES PADDING

The innovative padding ensures maximum comfort and reliable protection.

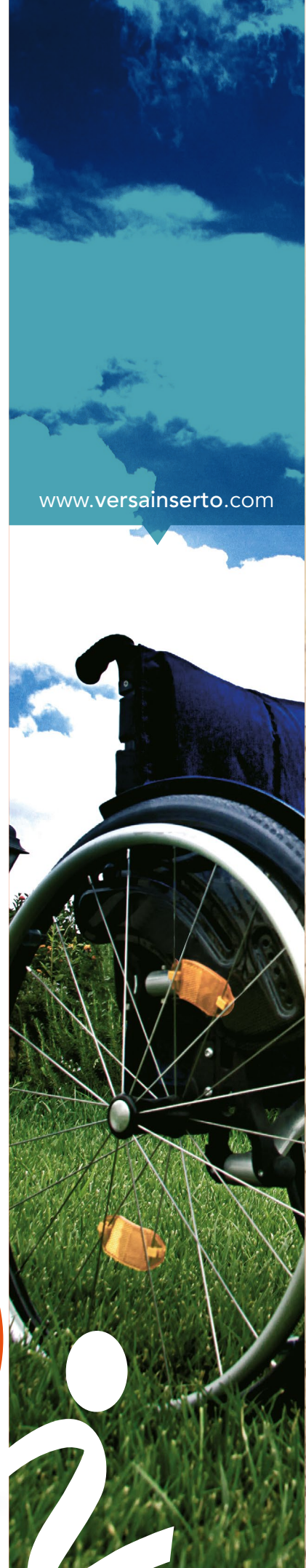



MANUFACTURED BY:



Via Montagna Z.I. - Lotto 41
72023 Mesagne (BR) - ITALY
Tel. +(39) 0831 777840
Fax. +(39) 0831 730739
Email: sales@promedicare.it
www.versainserto.com
www.promedicare.eu

CERTIFIED COMPANY WITH QUALITY
MANAGEMENT SYSTEM ISO 9001 / EN ISO 13485



ISO 9001
ISO 13485
BUREAU VERITAS
Certification



DISTRIBUTED BY:



CE 2 YEAR WARRANTY

www.versainserto.com