

## INSERTO SEAT VIP

This Technical Sheet is to be intended as an integral section of the Inserto Seat Range Instruction Manual. Before use, it is essential for the professional user to explain all procedures for a correct commissioning and a proper standard maintenance.

### 1. Commissioning

The **Inserto Seat Vip** pelvis positioning system of Inserto Seat Range comes as follows:

- 1) Different densities structural kit composed by a flat base of construction and 8 closed cells polyethylene inserts, modular and configurable, useful to support or correct the posture of the pelvis during the lifetime use
- 2) Padding kit composed by a superior layer of memory foam to provide the highest protection from decubitus sores, an inferior layer of flexible and elastic open celled foam and a pad of self-modeling polyethylene foam in the posterior part for increasing protection of the the ischium
- 3) An air-exchange, fireproof, antibacterial, latex free external cover, which is commonly used in medical devices and reduces the risk of skin irritation and an internal cover that covers the padding made of PU/PES combined material with the following properties: antibacterial, noncytotoxic, non-irritating, resistant to virus penetration, anti-fungal, fi reproof, biocompatible, OEKO-TEX® class I certified, latex-free, breathable and waterproof.



### 2. Use

The Pelvis Positioning System **Inserto Seat Vip** is recommended preferably, but not exhaustively for users with limited mobility with moderate deformities who need discrete postural containment and stabilisation, in particular:

- Those who have leg length discrepancies, wind-swept, pelvic obliquity, pelvis rotation, anterior and posterior pelvis tilt, adduction and abduction of the legs, of medium/severe entities;
- Those who need to stabilise and/or align ischium;
- Those who need ischial, trochanteric, coccygeal and sacral suspension;
- Those having high risk to develop decubitus sores;
- Those who need to balance and contain legs/pelvis/rachis/ relationship system .

The **Inserto Vip** pelvis positioning system can be combined, by means of an adhesive gripping tape, to any supporting base and/or wheelchair, provided that the structure will host the positioning system is solid enough to safely support the user during use.



*Once the modifications have been completed, such as removals, for the preparation and configuration as per prescription of the seat individually customised to the perfect reconstruction of the anatomic shape, the seat itself cannot be used by other users.*

The **Inserto Seat Vip** Positioning System , in its design integrity and numerical and dimensional completeness of the supplied components (structural kit, padding kit, internal and external cover), can be easily adapted to sizes/morphologies/deformities of the user. This kind of operations make the commissioning **referable to a serial manufacturing device**.

Alternatively, the commissioning of the **Inserto Seat Vip** Positioning System, deprived or modified, even partially, of its design integrity of numerical and dimensional completeness of the supplied components, built as per written prescription of a professional user in function of the anatomy/morphologies/deformities of the end user, through the measurement detection and direct trials, can be **referable to a custom-made device**.




Pic. 1: Pre assembled structural kit

## Inserto Vip

8 MODULAR AND CONFIGURABLE INSERTS



Pic. 3: Pre assembled structural kit + Padding kit

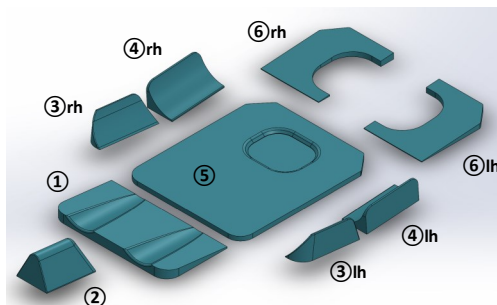

Pic. 2 Kit of inserts

- 01) Leg levelling insert
- 02) Abductor insert
- 03) Rh/lh adductor insert
- 04) Rh/lh pelvis guide insert
- 05) Flat base of construction
- 06) Ischial/coccygeal/trochanteric suspension rh/lh insert

The structural kit of **Inserto Vip** is composed by a flat base of construction and numerous inserts that can be customised, shaped, modified as needed (pic.1 and pic.2). Each element of the structural kit is supplied with male/female hooks and loops tape that strongly fix the inserts to the flat base of construction. The preparation of the pelvis positioning system shape, based on the prescription and the end user's anatomy and morphology, is done through the detection of measurement and direct trials, therefore it has to be carried out as follows in order to ensure the proper commissioning.

### 3. Suggestions related to some operations

#### 3.1 Positioning the gripping Tape

If it is necessary to replace or add the gripping hook and loops tape on one or more inserts of the structural kit, please use the extra tape supplied in the packaging.

To do this operate as follows: remove the adhesive film and stick the tape firmly on the insert to attach. Therefore, check for the correct placement. If it is correct, please remove the tape, taking care to heat the adhesive part for few seconds by using an industrial hot air dryer at temperature of around 100° (212°F), then reposition the tape definitely.



*Be careful not to damage the materials during this operation*

#### 3.2 Modifying the Structural kit Inserts

Modifications to reduce inserts by removing material, where necessary, can be carried out using a cutter.

If it is necessary to rebuild the modified component (where the material removed makes it possible), use an industrial hot air dryer at a temperature of around 100 ° (212°F) to heat and weld together the two parts to be assembled.



*Be careful not to damage the materials during this operation*

- 1) Accurately detect the measurements of the end user and of his wheelchair/mobility device
- 2) Remove the cover
- 3) Remove the padding

4) Dispose the pelvis guide inserts (4) on the flat base of construction (5) to match the width of the pelvis with the width of the seat; for this purpose, measure the width of the user's pelvis and modify the position of the pelvis guide inserts (4) so that the distance between their extremities matches with the user's width.

It should be noted that the function of the flat base of construction (5) is to provide a supporting base between the wheelchair/mobility device, as well as the primary structure of the construction kit on which to build or adapt the positioning system by using, removing or modifying, as necessary and as required by the prescription, all other inserts supplied with the construction kit.

#### Indications for the width of the seat

It is possible to reach the desired seat width by operating only on the flat base of construction (see point a), or on the flat base of construction (5) and consequently on the inserts of the structural kit (see point b), or on the inserts of the structural kit only (see point c).

a) **Flat base of construction:** The flat base of construction (8) has its own sizes accordingly to the size of the positioning system chosen. In order to insert the base on the cloth of the seat and reach the consequent compatibility with the width of the wheelchair/mobility system, it is possible to remove a portion of material from the sides of the flat base up to total 2 cm. (0.78") by using a cutter.

b) **Flat base of construction and consequently the set of inserts supplied:** After having carried out the operation of point a) it may be necessary to make dimensional adjustment of the width of the individual insert supplied with the structural kit, by removing the necessary material using a cutter. It is suggested to remove a small portion of material from the outer edges (max. 1 cm. -0.39") in order to avoid affecting the design. Alternatively it can be suitable the placement of the inserts on the flat base of construction by removing the exceeding inserts; alternatively both previous operation can be adopted. It is suggested to keep the part of material removed if not damaged by removal, occasionally, afterwards it may be reused in order to adapt the device to the modifications made to the user.

If it is necessary to increase the useful seat width with respect of all sizes defined for each model, it is possible to place each insert of the structural kit useful for the perimeter containment of the user up to 1 cm (0,39") out of the flat base of construction.

c) **Set of inserts of the structural kit:** It also may be appropriate to operate a dimensional adjustment of the width of each insert of the structural kit by removing the necessary material using a cutter. It is suggested to remove a small portion of material along the outer edges (max. 1 cm. - 0.39") in order to avoid affecting the design. It is suggested to keep the part of material removed if not damaged by removal, occasionally, afterwards it may be reused in order to adapt the device to the modifications made to the user. Alternatively it can be suitable the placement of the inserts on the flat base of construction (5) by removing the exceeding inserts; alternatively both previous operation can be adopted. If it is necessary to increase the useful width of the seat with respect of all sizes defined for each model, it is possible to place each insert of the structural kit useful for the perimeter containment of the user up to 1 cm (0,39") out of the flat base of construction.

5) Dispose the inserts to match the length of the femur with the length of the seat obtained by combining them.

It should be noted that the function of the flat base of construction (5) is to provide a supporting base between the wheelchair/mobility device, as well as the primary structure of the construction kit on which to build or adapt the positioning system by using, removing or modifying, as necessary and as required by the prescription, all other inserts supplied with the construction kit.

#### Indications for the length of the seat

It is possible to reach the desired seat length by operating on the flat base of construction only (see point a), or on the flat base of construction (5) and consequently on the set of inserts supplied in the structural kit (see point b), or on the inserts of the structural kit only (see point c).

a) **Flat base of construction:** The flat base of construction (5) has its own sizes accordingly to the size of the positioning system chosen and is shaped at the rear. Therefore it is possible to make it slide between the backrest tubes of the wheelchair/mobility device, once it has been positioned on the cloth of the seat, in order to reduce its depth up to 2/3 cm.

If the length achieved is not enough and it is preferable to maintain the entire length of the flat base or the end user has an irregular morphology and anthropometric measurements (i.e. leg length discrepancy), it is possible to reduce the effective length of the flat base as needed, by removing the portion of exceeding material with a cutter horizontally along the entire front or a part of it so as to make it asymmetrical.

b) **Flat base of construction and consequently the set of inserts supplied:** Having performed as necessary all or some of the operations described in (a) above, it may be advisable to make a dimensional adjustment of the depth measurement of the individual inserts supplied with the structural kit by removing the necessary material using a cutter. If it is necessary to reduce the depth of the pelvis guide inserts remove only a small part of material, horizontally.

If it is necessary to reduce the depth of the leveling insert remove only a small part of material, horizontally along the front edge, taking care to reproduce the rounded side profile.

It is suggested to keep the part of material removed, if not damaged after removal, in fact it can be reused to adapt the device to user's changes. Alternatively it can be suitable a placement of the inserts on the flat base of construction by removing the exceeding inserts; alternatively both operations can be adopted. If it is necessary to increase the useful length of the seat depth the leveling insert can be placed longitudinally along the flat base of construction. For this purpose, a measurement up to 2,5 cm. (0.98") longer than the actual length measurement of the flat base of construction can also be obtained. This can be achieved by positioning longitudinally the leveling insert on the flat base of construction, with their front portion out of it for a maximum of 2,5 cm. (0.98") and the rear portion connected to the surface of the flat base by a hooked gripping tape.

c) Set of inserts of the structural kit: It may be appropriate to make a dimensional adjustment of the measurement of depth of the individual inserts supplied with the structural kit by removing the necessary material using a cutter. If it is necessary to reduce the depth of the pelvis guide inserts remove only a small part of material, horizontally. It is suggested to keep the part of material removed, if not damaged after removal, in fact it can be reused to adapt the device to user's changes. Alternatively it can be suitable a placement of the inserts on the flat base of construction by removing the exceeding inserts; alternatively both previous operation can be adopted. If it is necessary to increase the useful length of the seat depth the leveling insert can be placed longitudinally along the flat base of construction in order to reach the desired total length. For this purpose, a measurement up to 2.5 cm (0.98") longer than the actual length measurement of the flat base of construction can also be obtained. What above can be achieved by placing the leveling insert longitudinally along the flat base with the anterior portion out of it for a max. of 2,5 cm. (0.98") and the posterior portion attached to the flat base through a hooked gripping tape.



*Please be aware that any depth customisation of the kit must be made by considering the harmony of the support and the compatibility with the wheelchair/mobility device with particular reference to the variables to the height and inclination of the footplates, depth of the seat cloth, inclination of the seating plan*

#### 6) Place the other inserts as necessary



*NOTE: The relation of the thrust, levelling and adhesion to the user's morphology exerted by the combined and harmonious use of each insert, enables the alignment and the postural compensation, as well as the distribution of the body loads along all the sitting surface. Use any useful insert among those supplied in order to achieve the compensation, support and posture correction and the individual seat most suitable to match the anatomic shapes of the user.*

#### 7) Cover the structural kit with the padding by adjusting it as shown in Pic. 3.

#### 8) Place the cover.

9) Once the pelvis positioning system has been assembled, have the user to be seated at least for one hour and verify if the new seat is causing pressure redness on the skin. If this happens it is recommended to adopt the most suitable interventions in accordance to the specifics defined for the user by the professional user under his sole responsibility. On a contrary case, instead, proceed with the delivery of the product to the user

10) It is advisable to keep documentary records of each operation carried out, as well as to provide the user with any deprived/removed parts which can be useful for after delivery interventions and/or adjustments. Before to use the product the professional user has to show the procedures for a correct commissioning and standard maintenance.



*It is strongly recommended to periodically check the skin of the user in order to verify any risk of redness appearance.*

11) When all operations of preparation of the kit to the shape and measures of the user have been accomplished and the positioning system is ready to be delivered, it is possible to remove the excess of padding from the edges by using a cutter. Take care to follow the direction of the cut as in the original design.



*In relation to the modification, processing and / or adaptation operations carried out on the structural kit and padding, the upper surface of the cover could result larger compared to the dimensions of the seating configuration obtained. Take care to spread the surface well when the user is sitting, in order to avoid wrinkles*

#### 4. Maintenance and cleaning

It is recommended to perform a careful cleaning and a standard maintenance every 2 weeks and/or, if needed, check the pelvis positioning system in all its parts in order to avoid any inconvenience. Regarding the cleaning of the padding, even if there is no direct contact with the skin, it is suggested to use a damp cloth or a brush with natural bristles and tepid water (max 30°C – 86° F), with the addition of a light gentle detergent dabbing with a dry cloth and then away from heat sources or sunlight;



*The drying time is quite long; it is suggested to be equipped of an additional kit of padding.*

Both the external and internal cover are removable and can be washed as follows:

- hand wash and then air dry
- washing machine (max temperature 60°C – 140°F) with the addition of a gentle detergent, bleach free, then centrifuge at a low spin.

Drying machines are not recommended. The material does not require the use of disinfectants.

The internal cover can be sanitized with the following detergents/disinfectants: PERSIL EXPERT; coldzyme/HENKEL, 70% etanol, PERFORM/Schuelke, TPH PROTECT/Schuelke, MIKROZID AF liquid/Oktal pharma, DESCOGEN liquid/Antiseptica, MANORAPID/Antiseptica, PLIVASEPT/Pliva, INCIDIN PLUS/Ecolab. Sodium Hypochlorite– based products cannot be used.



*It is advisable to use a protective wrapping before inserting in the washing machine to prevent the cover from tearing. It is suggested to be equipped of an additional cover.*

For further information, please contact our technical-Sales Department at the following number:



**0831 777840**

INSERTO VIP SIZE TABLE	ADULT																	
	PAEDIATRIC																	
MODEL	3232	3236	3434	3438	3638	3642	3840	3845	4040	4045	4242	4245	4250	4545	4550	4848	4850	5050
Effective width (cm)	32	32	34	34	36	36	38	38	40	40	42	42	42	45	45	48	48	50
Max width achieved with adaptation (cm)	34	34	36	36	38	38	40	40	42	42	44	44	44	47	47	50	50	52
Min. width achieved with removal of material (cm)	30	30	32	32	34	34	36	36	38	38	40	40	40	43	43	46	46	48
Effective length (cm)	32	36	34	38	38	42	40	45	40	45	42	45	50	45	50	48	50	50
Max. length achieved with adaptation (cm)	34,5	38,5	36,5	40,5	40,5	44,5	42,5	47,5	42,5	47,5	44,5	47,5	52,5	47,5	52,5	50,5	52,5	52,5
Min. length achieved with rear shaping adjustment and removal of inserts (cm)*	29	33	31	35	35	39	37	42	37	42	39	42	47	42	47	45	47	47
Min. and max. range of anterior height of flat base of construction (5) + Leg leveling wedge (1) (cm)	3,5/4,5 (heights referred to the models 3232 and 5050)																	
Min. and max. range of anterior height of flat base of construction (5) + Pelvis guide inserts (4) (cm)	9/13 (heights referred to the models 3232 and 5050)																	
Weight of positioning system min/max: 0,9 kg/3 kg																		
Max load referred to odel 5050 (50 cm x 50 cm): 150 kg																		

\*possible and further reduction due to removal of material



Any operation of removal, preparation or adjustment for the specific user, on the basis of a prescription, have to be performed by a professional operator and those interventions get the device customised. The professional user has the charge and the responsibility to guarantee the efficacy and the performances of the device.



**MANUFACTURED BY:**

 **Promedicare**

Feel the ideal posture

**Pro Medicare S.r.l.**

Via Montagna Z.I. - Lotto 41 72023 Mesagne (BR) - ITALY

Tel. +(39) 0831 777840

Email: [sales@promedicare.it](mailto:sales@promedicare.it) - [www.promedicare.eu](http://www.promedicare.eu)